


Easyboot Edge Sizing

How to measure for the Easyboot Edge:

1. After a fresh trim, measure the width of the hoof across the bottom at the widest point.
2. Measure the length of the hoof from the toe to the buttress line of the heel. The buttress line is the farthest weight bearing point of the heel where the hoof wall ends. Do not include the heel bulbs in the measurement.
3. Compare your measurement with the size chart below. Please note that while most of our other boots have their own unique size charts, all of the new 2009 boots all use the same size chart.
4. Ideally, the length and width measurement will fit into the same size. If the length and width measurements indicate different sizes, select the larger size. If the width and length measurements are different by more than one size, your selected boot style is not recommended. Be sure to check your measurements against an alternative boot in the EasyCare line—another style may accommodate your horse's hoof shape better.

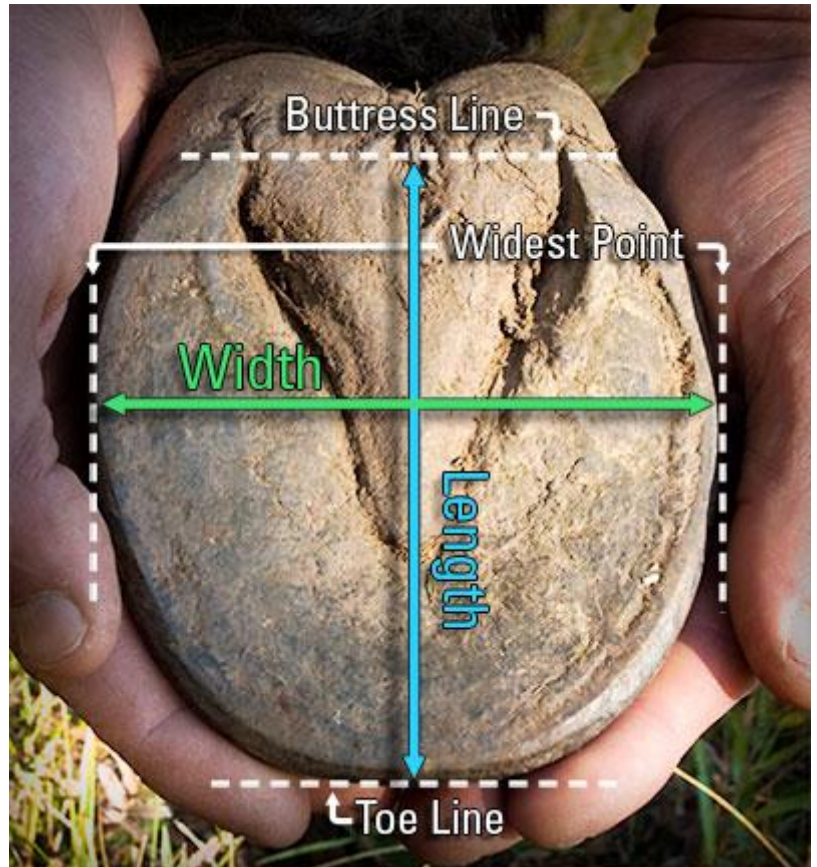

Easyboot Edge Size Chart

Size	Width in inches	Length in inches	Size	Width in mm	Length in mm
0	4 3/8"- 4 9/16"	4 5/8"- 4 15/16"	0	111-116mm	118-124mm
1	4 9/16"- 4 7/8"	4 15/16"- 5 1/8"	1	117-123mm	125-130mm
2	4 7/8"- 5 1/8"	5 1/8"- 5 9/16"	2	124-130mm	131-140mm
3	5 1/8"- 5 1/2"	5 9/16"- 5 15/16"	3	131-139mm	141-150mm

