


Easyboot Glove Sizing

Success of the Easyboot Glove is determined by accurate measurements and a snug fit.

How to measure for the Easyboot Glove:

1. After a fresh trim, measure the width of the hoof across the bottom at the widest point.
2. Measure the length of the hoof from the toe to the buttress line of the heel. The buttress line is the farthest weight bearing point of the heel where the hoof wall ends. Do not include the heel bulbs in the measurement.
3. Compare your measurements with the appropriate size chart. Ideally, the length and width measurements should fit into the same size on the chart. This boot needs to fit very snug. Be sure to check the requirements and measurements against an alternative boot in the EasyCare line—another style may accommodate your horse's hoof shape better.

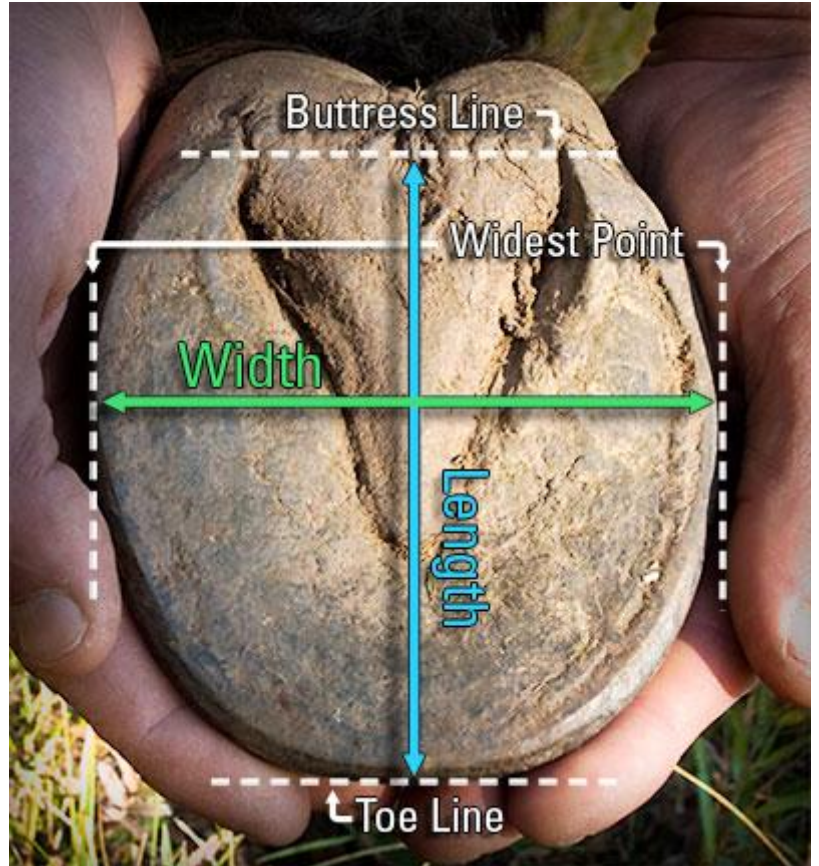

Easyboot Glove Size Chart*

Size	Width in mm	Length in mm
00	96-99mm	106-109mm
00.5	100-103mm	110-113mm
0	104-107mm	114-117mm
0.5	108-111mm	118-121mm
1	112-115mm	122-125mm
1.5	116-119mm	126-129mm
2	120-123mm	130-133mm
2.5	124-127mm	134-137mm
3	128-131mm	138-141mm

**Do not assume that, if you wear a certain size in another boot style, you will wear the same size Glove.*

